Cleaning and Care Instructions  
Instructions d'entretien et de nettoyage  
Istruzioni per la cura e pulizia  
Οδηγίες καθαρισμού και φροντίδας  
Instrukcje czyszczenia i pielęgnacji  
Návod k čištění a údržbě  
Instrucciones de cuidado y limpieza  
Anleitung zur Reinigung und Pflege  
Instruções de limpeza e cuidado  
Reinigings- en onderhoudsinstructies  
Instruktioner för rengöring och skötsel  
Temizleme ve Bakım Talimatları

## Montgomery® Speaking Valve

REF: \_\_\_\_\_ LOT: \_\_\_\_\_

### IMPORTANT:

Please deliver these important instructions to the  
Montgomery Speaking Valve patient / caregiver.

N-MVCC-2 – 2023-01

<b>English .....</b>	<b>3</b>
<b>Español .....</b>	<b>14</b>
<b>Français .....</b>	<b>26</b>
<b>Deutsch .....</b>	<b>39</b>
<b>Italiano.....</b>	<b>51</b>
<b>Português .....</b>	<b>63</b>
<b>ελληνικά.....</b>	<b>75</b>
<b>Nederlands .....</b>	<b>89</b>
<b>Polski.....</b>	<b>101</b>
<b>Svenska.....</b>	<b>113</b>
<b>Český.....</b>	<b>124</b>
<b>Türkçe .....</b>	<b>135</b>

## 1 Introduction

### 1.1 Dear Patient,

You have had inserted a speaking valve of the type *Montgomery Cannula Speaking Valve / Montgomery Tracheostomy Speaking Valve*.

Speaking valves have been used successfully to treat patients like you for many years.

However, care must be taken to keep the speaking valve clean and clear of mucus / secretions which, if left unattended, could eventually result in partial or complete blockage of the speaking valve.

Therefore, please read and follow these Cleaning and Care Instructions and save these instructions. If after reading these instructions you have any questions or concerns, please contact your physician's office, Boston Medical Products or your BMPI Distributor.

### 1.2 Symbols Glossary

Symbol	Description
	Caution: Consult Instructions for Use

Symbol	Description
	Do not use if package is damaged
	Keep away from direct sunlight
	Keep dry
<b>REF</b>	Catalog number
<b>LOT</b>	Batch code
<b>QTY</b>	Quantity per packaging unit
	Manufacturer
<b>EC REP</b>	(EU) Authorized representative in the European Community
<b>R<sub>x</sub> ONLY</b>	(USA) Caution: Federal Law restricts this device to sale by or on the order of a physician.

*Table 1: Symbols Glossary*

### 1.3 Safety Information Marking

#### **WARNING**

Non-compliance may result in serious injuries, serious deterioration of your general condition or your death.

#### **NOTICE**

Product damage or other damage may occur in case of non-compliance.

## 2 Important Safety Information


#### **WARNING**

- Read these instructions carefully, adhere to them and save them.  
Otherwise there are risks to your health.

#### **NOTICE**

- Handle the product with utmost care.  
Otherwise, the product could be damaged.

### 3 Your Speaking Valve

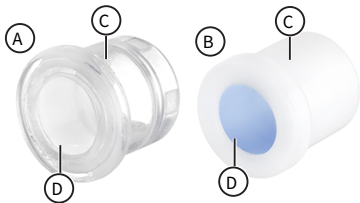- A Montgomery Cannula Speaking Valve
- B Montgomery Tracheostomy Speaking Valve  
(not available in all markets)
- C Body
- D Diaphragm

These items are components of your speaking valve and need to be handed to you from your physician:

- 1 x speaking valve
- 1 x storage container
- 1 x Cleaning and Care Instructions (N-MVCC)

#### **4 Prior to Leaving the Clinic**

The production of mucus and secretions varies between the patients. Also, the condition of the skin around the tracheostoma differs from patient to patient.

Therefore please check with your physician prior to leaving the clinic:

- How often do you need to apply these cleaning techniques?
- Where do you obtain the material required for the cleaning techniques?
- How and how often do you have to take care for the skin around your tracheostoma?

#### **5 Cleaning and Care**

The manufacturer recommends to have 2 products on hand. This allows one product to be used while the other product is being cleaned.

Required equipment and materials:

- Warm soapy water
- Cotton-tipped applicators

## 5.1 Removing the Speaking Valve

### 5.1.1 Montgomery Cannula Speaking Valve

1. Removing the speaking valve:  
Grasp the tracheal cannula with one hand. Twist the speaking valve slightly and remove it from the tracheal cannula.

### 5.1.2 Montgomery Tracheostomy Speaking Valve

1. Twist the speaking valve slightly to remove it from the 15 mm adaptor.

**ATTENTION:** Do not twist the speaking valve into the direction that is needed to remove the inner cannula but twist it into the opposite direction.

In case the inner cannula accidentally comes out: Re-insert it.

## 5.2 Cleaning the Speaking Valve

1. Soak the product for 10 minutes in warm soapy water.

2. Clean the inside of the valve using a cotton-tipped applicator.

**ATTENTION:** Be careful not to damage the delicate silicone diaphragm.

3. Rinse the product under running water until the detergent is completely washed away.
4. Allow the valve to dry.

### **5.3 Attaching the Speaking Valve:**

#### **5.3.1 Montgomery Cannula Speaking Valve**

1. Attaching the speaking valve:  
Grasp the tracheal cannula. Hold the speaking valve at a slight angle against the tracheal cannula. Push the speaking valve onto the tracheal cannula with a slight twist. When you feel the speaking valve snapping into place it is ready for use.

### 5.3.2 Montgomery Tracheostomy Speaking Valve

1. Press the speaking valve onto the 15 mm adaptor without using excess force. In doing so, twist the speaking valve slightly.

## 6 Warranty

The reliability of the product's material and design at the time of shipment is guaranteed. The manufacturer does not know either the diagnosis of the patient or the nature of the application and has no influence on the conditions under which the product is used. The storage conditions after delivery of the product are also beyond the manufacturer's area of responsibility. Due to biological and individual differences, no product is 100% effective under all circumstances.

**Therefore, the manufacturer cannot guarantee a positive effect or the absence of negative effects for product application. The medical staff must use the product on the basis of their medical training and experience, and they are responsible for correct application.**

The warranty (repair or replacement) applies only if the product is used in accordance with these Instructions for Use (for instruments, particularly with regard to handling, cleaning, sterilization and maintenance); the warranty period starts on the delivery date.

If you have reason to believe that a new product is faulty, please contact the Customer Service in writing immediately and provide as detailed a description as possible of the fault, the REF (product code), and the LOT (batch code) and/or series number. All allegedly defective products must be returned to us for inspection. Instruments have to be completely cleaned and sterilized, appropriate documentation must be enclosed with the return.

If the manufacturer finds that despite all due care the product was defective at the time of delivery, he will repair the product or replace it promptly. If repair or replacement of the product is not possible, the buyer has the right to cancel the purchase or to reduce the payment, but by a maximum of the purchase price amount.

Additional claims or those not mentioned here due to defect, and other claims regardless of the legal reason, including those based on illegal acts and for compensation of immaterial damages against the manufacturer, his agents, dealers and suppliers, are excluded unless existing law is contrary to the liability exclusion, e.g. in cases of intent or gross negligence or in the event of physical injury.

All claims based on the consequences of non-compliance with the Instructions for Use, including specified indications, contraindications, warnings, instructions, application, storage and off-label use, as well as the consequences of a combination with third-party products are excluded.

Furthermore, all claims that result from the use of products that have expired, or were used despite the obvious damage to the packaging, or resterilized and/or recycled contrary to the Instructions for Use, are excluded.

No one is allowed to change the above conditions, make further warranty or liability declarations, or guarantee any properties that surpass those specified in the Instructions.

The General Terms and Conditions of the manufacturer, which can be accessed at <http://www.bosmed.com> apply in all remaining instances.

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## Boston Medical Products, Inc.